WU/MVE-Serie

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Bed head Units for Medical Applications

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**Instruction for Use / Installation
Instruction**

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WARNING!

For a full understanding of the performance characteristics of this device, the user should carefully read these instructions before use of the device.

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WARNING!

Important information concerning a potentially dangerous situation, which can cause fatalities or serious injuries, if not prevented.

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CAUTION!

Important information concerning a potentially dangerous situation, which can cause minor or moderate injuries for the user or patient resp. damage to the medical device or to other objects, if not prevented.

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NOTE:

Additional information, to provide an easier understanding of the use of the medical device.

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For Your Safety and that of Your Patients

Please follow Instructions for Use carefully

WARNING

Every use of the medical device requires full understanding and strict observation of these Instructions for Use. The medical device is only to be used for purposes specified in the "Intended Use" section on page 7. All references marked with **WARNING** or **CAUTION** in these Instructions for Use, as well as the references on the medical device labels must be observed carefully.

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Accessories

WARNING!

The given maximal loads and moment are to be observed when using accessories on the standard rails.

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Maintenance

NOTE:

The wall duct does not require regular inspection or maintenance.

A regular inspection of gas – electrical components is to be carried out through the operator.

For the respective systems, the inspections will include the various components of the medical gas supply system (terminal units), electrical supply (outlets) and data technology (low-voltage supply).

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WARNING!

The medical device must not be operated or used during maintenance.

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Patient Safety

The instructions for use, including the 'WARNING' and 'CAUTION' references are essentially limited to the special features of the medical device.

Alterations to the medical device or incorrect use of the medical device can be dangerous.

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Liability for Proper Function or Damage

The liability for the proper function of the device is irrevocably transferred to the owner or operator to the extent that the device is not installed correctly by professional experts.

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VTS cannot be held responsible for damage caused by non-compliance with the recommendations given above. The warranty and liability provisions of the terms of sale and delivery of VTS are likewise not modified by the recommendations given above.

VTS Vision Technology Systems
GmbH

VTS Vision Technology Systems
GmbH

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Intended Use

The medical supply units of the WU and MVE series are wall-mounted supply units and are used to pass electrical energy, medical gases and to provide indirect room lighting and reading light.

In addition, the medical supply units of the WU and MVE series provide space for the installation of communication systems, such as a nurse call. Furthermore, the purpose of the medical supply units of the WU and MVE series is to place further medical equipment on the standard- and equipment-rails.

Medical supply units are not designed for dedicated patient groups.

WARNING!

Only the operation of the light may be carried out by the patients after instruction by the operating personnel. All medical-technical functions must not be carried out by the patient.

User groups Clinical users

This user group uses the product in accordance with the intended use.

Users are medical experts and patients as well as, if applicable, visitors to patients in continuous operation.

Service personnel

This user group installs the product and performs service activities.

If product-specific skills or tools are required, then the service activities must be performed by specialized

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service personnel. Therefor, qualified specialists for assembly/ commissioning and ongoing cleaning and maintenance are additional users.

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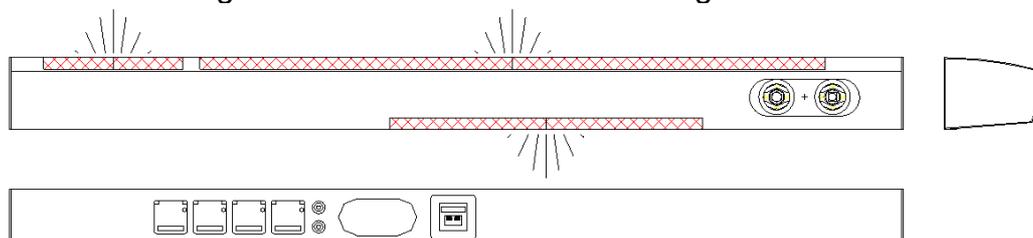
Description

Medical treatment rooms supplied with lighting, medical gases and electrical connections.

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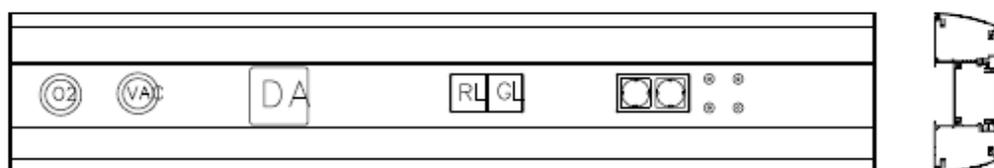
Versions

MVE – Living

Bed Head unit with an outlet duct and direct and indirect lighting, depending on design.

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WU/MVE – Normal Care

Bed Head unit with an outlet duct as well as direct and indirect lighting, depending on design.

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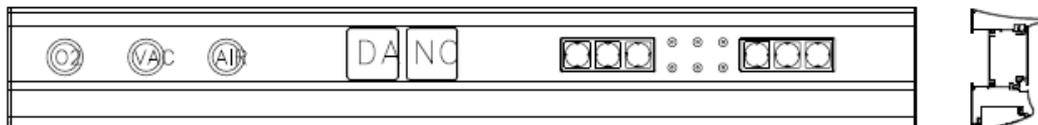
WU/MVE – Intermediate Care

Bed Head unit with an Service compartment as well as direct or indirect lighting and standard rails integrated into the top or bottom of profile, depending on design. In case of use of the medical rail integrated in

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the profile, there's no lightning option

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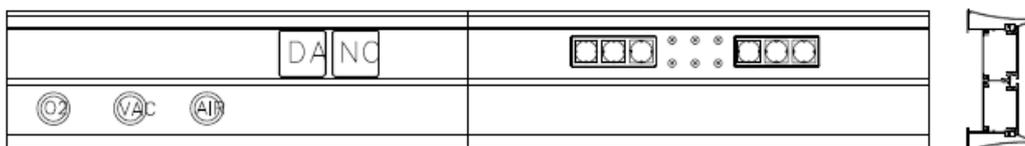
WU/MVE – Intensive Care

Bed Head unit with two service compartment and two standard rails integrated into the profile, depending on design.

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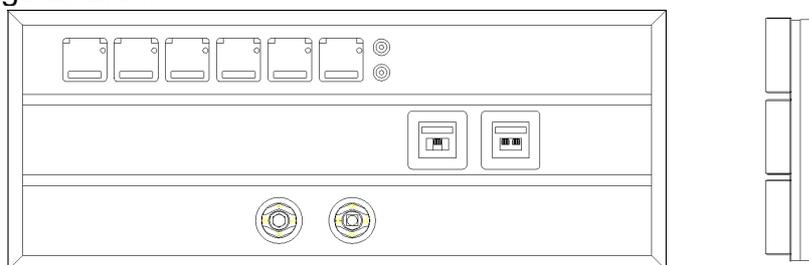
WU – Flush

Bed Head unit for wall installation with an Service compartment.

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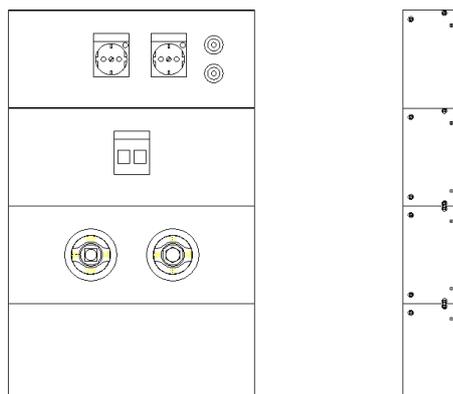
WU – Trunk

Bed Head unit with an Service compartment.

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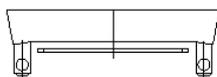
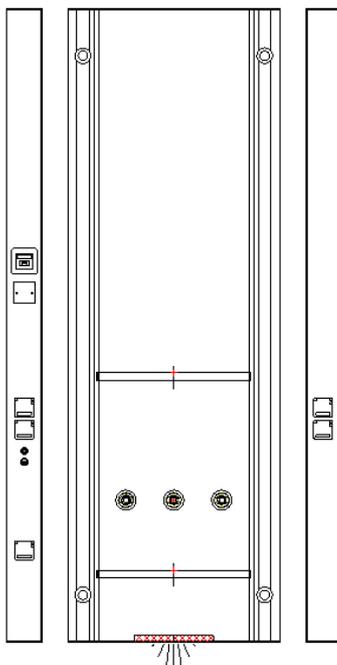
WU/MVE – Vertica

Bed Heat unit with two sidepanels and
front installation.

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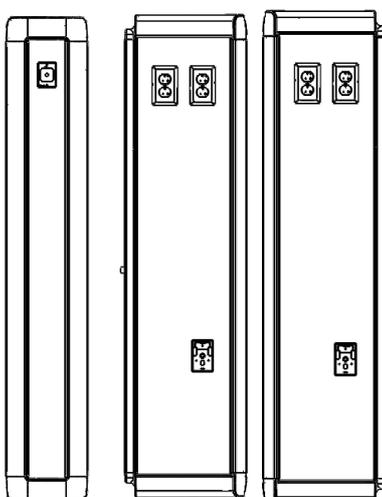
WU – Kolona

Bed Heat unit with three sidepanels
and possible lightning.

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Load Capacity

WARNING!
**Danger of personal- and material
damage when exceeding the
load capacity!**
**The system maximal load must
not be exceeded.**
**Pay careful attention to maximal
load!**

The maximal load onto standard rails is 50 kg per meter of rail resp. 12,5 or 25 kg per device when connected directly to the standard rail.

The load on the round tubes Ø38 mm of the Vertica is 100kg.

A maximum load of 160kg is possible with the Kolona

For accessories with extension arms resp. accessories with levering affect, the maximal moment of load capacity on standard rail is 5 Nm.

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Non-permissible Use

WARNING!

Objects placed on the wall duct can fall off (e.g. cups containing liquid; writing materials, clothing, etc.).

Risk of injury!

Risk of scalding with spillage of hot liquids!

Risk of short circuits!

The wall supply rail is not allowed to be used for placing objects on.

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Optional:

Instead of a low-voltage handset control, the MVE is also available with with a pull cord switch for light control.

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Pull Cord

WARNING!

A dangling pull cord can be a safety hazard, in particular for children.

Risk of strangulation!

Special care is to be taken during patient supervision!

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NOTE:

For hygienic reasons, the pull cord is to be cleaned/disinfected regularly.

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Operation

WARNUNG!

O₂ and nitrous oxide are easily flammable gases.

Increased fire hazard!

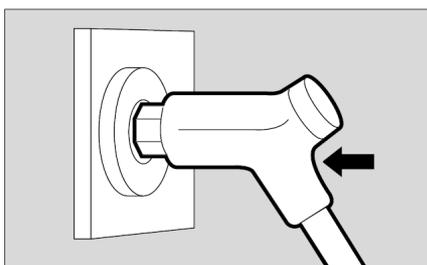
Do not bring terminal units and connectors for O₂ and nitrous oxide in contact with oil, grease or flammable liquids.

Gas supply

- Ensure that the probe identification matches the release bushing of the terminal unit.
- Ensure that the gas supply tube of the probe is connected tightly to the medical device.

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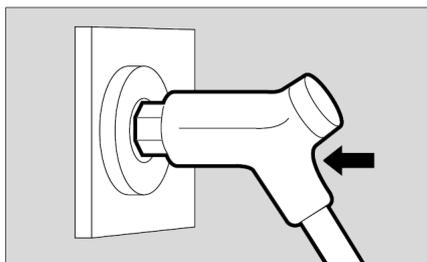
Probe in Parkposition

- Insert probe into terminal unit to first click-lock position.

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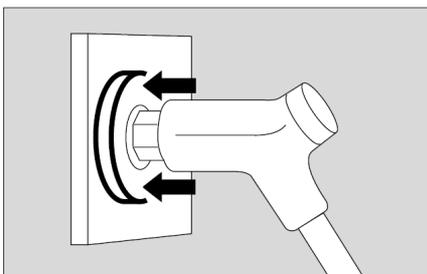
Gas supply

- Press probe into second click-lock position.

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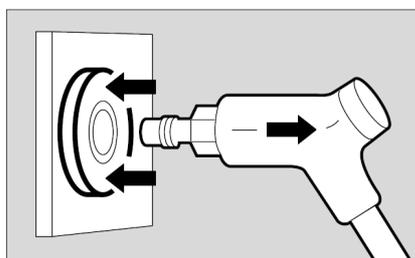
Interruption of Gas Flow

- Press release bushing lightly: the probe returns to the first click-lock position (park position).

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Disconnection

- Press release bushing **and simultaneously** pull probe out of the terminal unit.

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Cleaning/Disinfecting

WARNING!

Electrical mains!
Risk of electric shock.
Do not open the housing during cleaning process.

WARNING!

Penetration of liquids can influence the functionality of the device or can cause damage to the device.
Danger for patients!
Risk of short circuits!

Wipe-disinfect the parts with moist cloth only and ensure that no liquids can penetrate.

CAUTION!

Some cleaning agents release chlorine or oxygen.
Health hazard!
Do not use cleaning agents which release chlorine or oxygen.

CAUTION!

The device is partially made of materials, which are not resistant to some components of surface disinfectants.
Risk of surface- and material damage.
Do not use disinfectants which are based on phenols, halogen-releasing compounds as well as oxygen-releasing compounds!
Do not use solvents such as benzene or ether!

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Surface disinfectants which are based on aldehydes and quaternary ammonium compounds are listed in the current "German Society for Hygiene and Microbiology".

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Commissioning

The installation, testing and operation
of MEDICAL SUPPLY UNITS shall be
in accordance with ISO 7396-1:2007,
ISO 7396-2:2007 and the
MANUFACTURER'S instructions.

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Delivery

Inspect delivery
Check the shipping crate for damage
Ensure that the delivery is complete:
Instructions for Use / Installation
Instructions

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Required Tools and Additional Components

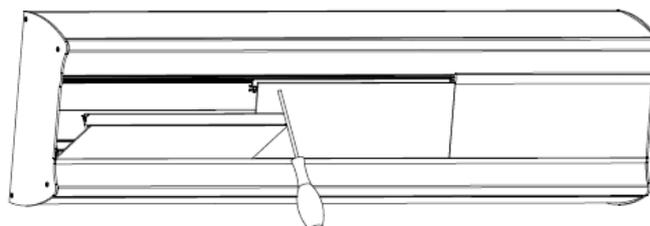
Drill
Level
Wrench
Screwdriver Torx 15, Torx 20
Fastening material

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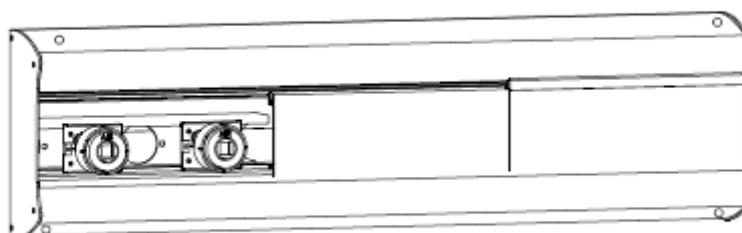
Installation

1. Double-check the drilling and
feed-through dimensions acc. to
enclosed diagram
2. Remove the rubber sealing lips,
loosen the screws of blind cover
above the feeding point; remove
plates and light cover.

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3. Mark **X** at drill hole for the wall-
mount acc. to enclosed diagram.

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WARNING!

Improper execution of electrical and gas installation can cause damage and injuries.

Critical danger! Risk of material damage!

Work on electrical- and medical gas supply is only permitted through trained experts!

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Initial Operation

CAUTION!

Prior to initial operation of terminal units and after exchange of compressed gas tubes in medical supply units, the 3 following tests are to be carried out in accordance with DIN EN ISO 7396-1

- Leakage test;
- Blockage test;
- Particle pollution test;

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- Testing the gas-type.
- If these tests do not pass inspection, the medical devices are not allowed to be put into operation.

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Final Test

(acc. to enclosed acceptance protocol)
The acceptance test includes:
Gas-type inspection
Inspection of electrical circuits (also for emergency power outlets, if applicable)
Ground leakage current

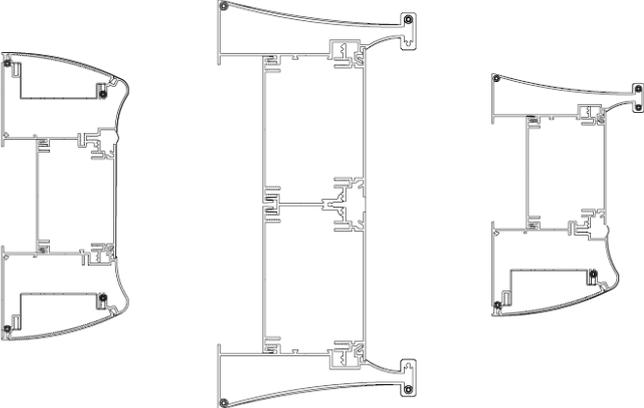
Disconnection from the mains supply

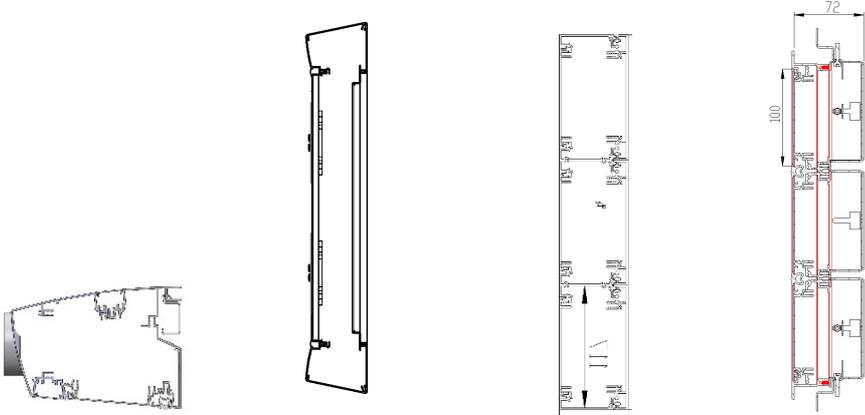
The medical device has no disconnection equipment from the mains supply, if disconnection is necessary, the locally applicable regulations of the electrotechnical specifications must be followed.

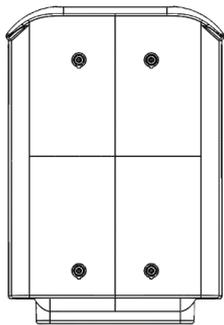
Final disposal

After deinstallation of the supply unit, the dismantled individual parts can be disposed of in the respective recycling circuits.

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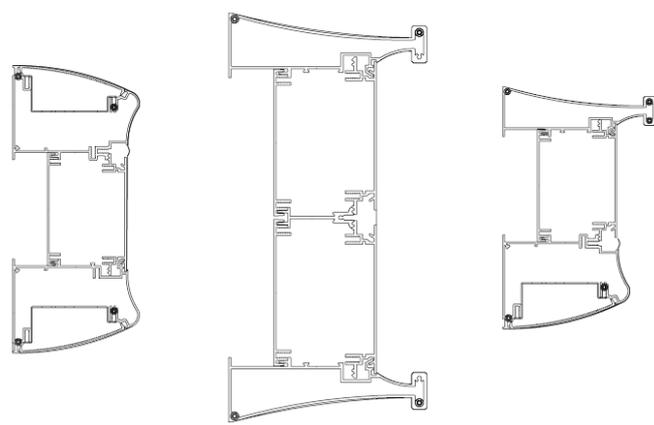
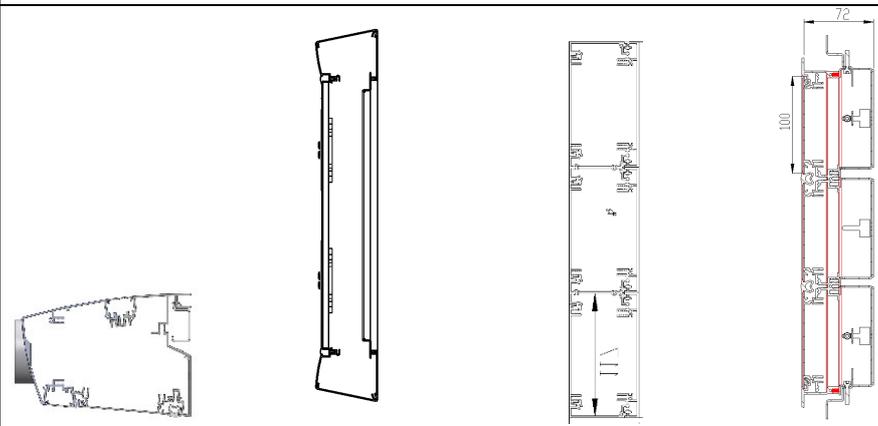
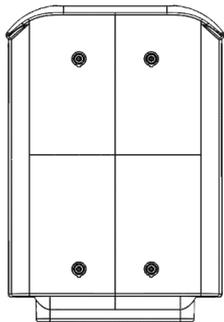
Technical Data WU/MVE

	WU/MVE Linea N	WU/MVE Linea I	WU/MVE Linea IM
Wall-mount	Via wall dowels. Max. moment of 250Nm per meter of duct. Bed Head unit either from back or side.		
Length	Any length available, up to 6m length		
Height / depth	250 mm / 115 mm	358 mm / 135 mm	213 mm / 135 mm
Cover plates	1-3	1-3	1-3
Outlet intervals	Between gas terminal units and electrical outlets, the minimal distance is 200mm. The minimal distance between two media outlets is 75mm.		
Profie cross-section			
Surface	RAL 9002; all other RAL-colours from the enclosed colour chart are available upon request		
Lighting	Indirect lighting LED indication light LED reading light LEDt	No lighting	Indirect lighting LED and indication light LED (top) reading light LED (bottom)
Load capacity	The standard rails have a max. loading capacity of 50kg per meter.		
Accessories	Rail on the top, rail on bottom.	accessories for standard rails.	

	Living LED Linea Living	Vertica Linea Vertica	Trunk Linea Trunk	Flush Linea Flush
Wall-mount	max. 65 kg/m, Lateral force per threaded bolt (M10) 500 N.	max. 65 kg/m, Lateral force per threaded bolt (M10) 500 N.	Mittels Dübel an der Wand; Drehmomente max. 250 Nm je Meter Kanal	Mittels Dübel an der Wand. Aufretende Momente max 250Nm je Meter Kanal.
Length	Up to 6000mm	Up to 4000mm	Up to 4000mm	Up to 4000mm
Height / depth	130mm/ 179mm	666mm/ 11009mm	broad 120mm/ depth 68mm	153mm/ 72mm
Cover plates	1	1	1-3	1-3
Outlet intervals	Between gas terminal units and electrical outlets, the minimal distance is 200mm. The minimal distance between two media outlets is 75mm.			
Profile cross-section				
Surface	RAL 9002; all other RAL-colours from the enclosed colour chart are available upon request			
Lighting	Indirect lighting LED indication light LED reading light LEDt	Night light LED	Night light LED	Night light LED
Load capacity	Die Normschiene sind mit 50kg max. Zuladung pro Meter belastbar	Geräteschiene: max. 50 kg pro Meter / Rundrohr: max. 100 kg pro Roh	Die optional integrierte Geräteschiene ist mit 12,5 kg max.Zuladung pro Schiene belastbar	No load
Accessories	accessories for standard rails			No accessories

Kolona	
Wall-mount	Wall must have a minimum load capacity of 40kg device and 160kg load
Length	1500 mm
Height / depth	237 mm / 346 mm
Outlet rows	3
Outlet intervals	Between gas terminal units and electrical outlets, the minimal distance is 200mm. The minimal distance between two media outlets is 75mm.
Profile cross-section	
Surface	Equipment plates anodized. All other RAL colors from the color table are possible on request
Lighting	Indirect lighting LED reading light LEDt
Load capacity	max. 160 kg
Accessories	Dräger accessories for standard rails.

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Technical Environment

Operating temperature:
+10°C to +40°C
Transport and storage:
-20°C to +60°C

Note: The specified environmental conditions must be observed.

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UMDNS 16-997

Directive 93/42/EEC for Medical
Devices
Annex IX Class IIb

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NOTICE:

Users and/or patients shall report all serious incidents related to the device to the manufacturer and the competent authority of the Member State.

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© **V.T.S. GmbH**
Edition: 09-2021
V.T.S. reserves the right to make
modifications to the medical device
without prior notice.

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Vision Technology Systems GmbH